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Arab Republic of Egypt  
Egyptian Drug Authority  
CA for Medical Devices  
General administration of Medical  
Device Registration  
administration of local medical devices  
registration

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### Packaging

Description configuration	single unit packed in medical paper and PE/PP roll, packed 25 in PE bag and packed 500 Pcs outer carton
-Material of primary Pack	PE/PP roll & Medical paper
-No. of units per primary pack	1
-Material of Secondary Pack	PE
-No. of units per Secondary pack	25
-Material of Tertiary Pack	Carton
-No. of units per Tertiary pack	500
Storage Condition	Shall be stored in cool & dry place. -Do not exposure to heat or direct sunlight. -the Temperature limit from 15°C to 35°C. -Keep away from any source of ignition
Country of origin	Egypt Free Zone

### Manufacturer information

Manufacturer	Euromed For Medical Industries	Free Zone-Nasr City-Part no.10-Block (I)
--------------	--------------------------------	--

### Certificate information

#### CE certificate

Certificate no.:	Notified Body name:	Notified Body ID:	Issue date	Expiry date
EG20/2833	SGS Belgium NV	1639	28/10/2020	24/05/2024

#### ISO 13485:2016

Certificate no.:	Notified Body name:	Issue date	Expiry date
EG07/72442	SGS united kingdom Ltd	01/08/2022	16/07/2025

#### Free Sale Certificate

Certificate no.:	Issuance Authority:	Issue date:	Expiry date
PS/CLV/EBL/1158/2022-CEX	Spain	14/09/2022	—

#### Variant

Variant	(I.V 009) Euro-Infusion set with flash back 100cm/ (I.V 010) Euro Infusion set with flash back 110cm/ (I.V 011) Euro-Infusion set with flash back 120cm/ (I.V 012) Euro-Infusion set with flash back 125cm/ (I.V 013) Euro-Infusion set with flash back 140cm/ (I.V 014) Euro-Infusion set with flash back 130cm/ (I.V 015) Euro-Infusion set with flash back 150cm
---------	---

### Raw Material

Component	PVC Tube	Polyvinylchloride PVC medical grade
Component	spike cap	Polypropylene "P.P" medical grade
Component	spike	Acrylonitrile butadiene styrene "ABS" medical grade
Component	Drip chamber	Polyvinylchloride PVC medical grade
Component	Clamb roller	Polypropylene "PP" medical grade)
Component	Clamp body	Polypropylene "PP" medical grade
Component	Slip connector	Acrylonitrile butadiene styrene "ABS" medical grade
Component	Hypodermic needle	Stainless Steel304
Component	Flash back	latex free Rubber

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Non-toxic, Pyrogen free.

### General Description:

Convenient, easy to use, sharp spike with flexible drip chamber  
 Clear transparent and flexible drip chamber to see flow rate and allows an estimation of flow rate. Accurate flow control  
 Strong yellow clamp & wheel with provision to hold the tube for changing the fluid container without disturbing the flow rate.  
 Kink free & 100 cm transparent PVC tubing for easy visualization of flow rate, taper foot, as per International Standard.  
 Flush ball type latex free stopcock mechanism. Also available with a Y injection site (with or without  
 Luer)

Options of luer lock connector, needle available. Single use and non pyrogenic

### Intended Use:

Surgically invasive short term use  
 Used for the infusion set of T.V. fluids and fluid distribution.  
 Infusion sets are used for the controlled infusion of medications, typically over long periods of time. Infusion sets are used to  
 connect the medication to the needle inserted into the patient. Infusion extension sets are also used to extend  
 I.V. lines without risk of contamination.

### Indications:

Intravenous Infusion (Infusion set) is the fastest way to deliver medication or replace fluids throughout the body. Starting  
 infusions are solutions are administered into a vein through an infusion set that includes a plastic/glass vacuum bottle or bag  
 containing the solution to be administered.

### Intended Patient Population:

This product is used for pediatric patient and/or heart patient or kidney patient, also used for patients who need  
 small drops and small medicine and short interval.

### INSTRUCTION FOR USE:

- 1- Close the roller clamp
- 2- Remove spike protection
- 3- Insert spike into infusion set container
- 4- perform connections
- 5- Open clamp and permit fluid to flow in drip chamber & tubing. 6- Control fluid flow through roller clamp (flow regulator).
- 7- Close roller clamp (flow regulator) after finishing fluid transfer.
- 8- Medication may be added through latex injection site at end of tube. 9- Product is single use only, discard the product in safety  
 box.

### Sterilization:

Infusion set is sterilized by Euromed for Medical Industries, EO sterilization, biological indicator testing and aseptic process  
 are performed by Euromed for Medical Industries Company.

### Shelf life:

Following certification the product has 3 years shelf life, provided that it is stored in the original packaging in recommended  
 storage condition and that the packs are not damaged or exposed or tampered with in any way.

### WARNINGS:

Product is single use only. Re-use of the product can cause change in physical characteristics.  
 The product should be checked for any major complications i.e. In front of spike Body clamp, flexible tubing, Injection port,  
 drip chamber, injection site, Luer lock connector, S.S Needle  
 The set so easily may increase the EO residual in the product or physical property of the products may be change.  
 Visually inspect and carefully check the product and packaging before use.  
 Improper transport and handling may cause structural/functional damage to  
 device or packaging.  
 Temperature / Humidity of the material should be done under covered. It will contain to avoid any damage, rain & direct  
 heat.

### Read instructions before use.

Prevent proper disposal of product and packing after use.  
 Use the product immediately after opening the packaging please destroy when it was used.  
 This product should be used by a Doctor / Registered Practitioner or Paramedic.

### Precautions:

Read instructions before use.  
 Check expiry date prior to use.  
 Examine before use that product / packing is not ruptured or damaged.

### Contraindications:

Product use with patients with known hypersensitivity to any of the materials used.  
 Administration of high viscous fluids.  
 Large blood transfusion.

REF Product Code

LOT Lot No./Batch No.

EC REP CMC  
C/Hispania Lengua 19-18,  
CP 29006, Málaga Spain

STERILE EO Sterilized By Ethylene Oxide

Date of Manufacturing

Use by / Expiry Date

Do Not Reuse

Do Not Re-sterilize

Non-Pyrogenic

Do Not Use if  
Package is Damaged

Latex Free

Caution

Consult Instructions For Use

Phthalate Free

Storage Condition

### M.O.H Reg. No.

REF 1.V 015

B.N 22/146

LOT 22/306

09-2022

08-2027

EC REP CMC  
C/Hispania Lengua 19-18,  
CP 29006, Málaga Spain

Euromed For Medical Industries.

Area No. (10), Block (j),

Nasr City, Cairo - Egypt

Tel. : 202 22 72 59 52/3 Fax : 202 22 72 93 74

E-mail : euromed@euromed.com.eg

Website: www.euromed.com.eg

CE 1639

F/QP-25-0142/REV.01  
30/11/2021

Quality Manager  
Shimaa Rabeey

.....

 <p>F/OP-25-201/RW/01 30/11/2021</p>	 <p>Cautioned For Medical Industries, Area No. 108 Block B, El-Dokki, Giza, Egypt Tel: +201 2203 74000 Fax: +201 22279154 E-Mail: <a href="mailto:com@euromed.com.eg">com@euromed.com.eg</a></p>		 <p><b>Euro-Infusion Set with flash back</b> STERILE 600 PCS Made In Egypt</p>
---	--	---	---

**Quality Manager**  
**Shimaa Rabee**